

Emu-Graph 7

Chemical Emulator

CI 142

PACK RELEASE

EMU-GRAPH™ 7 STEAM
ISO 11140-1 TYPE 6 7 MIN/134°C - 20 MIN/121°C
SteriTec LOT 0000-00 ACCEPT IF GREEN


INTERPRETATION

*Note: Any purple remaining indicates a fail



Unexposed

Indicator line changes color from purple to green when all parameters for sterilization are met

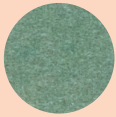

Correct color change



Insufficient

Any residual purple color left in the line on the indicator indicates a fail

Provides an integrated response to steam sterilization in pre-vacuum sterilizers.

- Vivid purple to green color change
- Complete color change in 7 minutes at 134°C or at 121°C for 20 minutes
- Full cycle monitor that is sterilant specific meaning it must have steam to change
- Lead and toxic heavy metal free

Conforms to **ISO 11140-1, Type 6**

Pack release common failures:

- Inadequate cycle time
- Improper cycle selection
- Overloading the sterilizer
- Poor steam quality
- Equipment malfunction
- Blocking indicator with large metal mass

Ordering Information

Part No.	Case Qty.
CI 142	20 bags of 250



MEMBER OF THE GETINGE GROUP

WASH RELEASE MACHINE RELEASE LOAD RELEASE PACK RELEASE
www.steritecproducts.com ✨ info@steritecproducts.com ✨ youtube.com/steritecproducts

MK-226-9 Rev. A